AirPolish

Prophy Jet Powder, Mint Flavor

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Production License No:
52705376

DESCRIPTION:

Air polish is a white, flavored free flowing powder for professional prophylaxis procedures. Used with an air polishing system, the Air polish provides effective plaque and stain removal for shade determination and placement of sealants. It is also used prior to fluoride treatment or whitening procedures; surface preparation prior to bonding, cementation of inlays, onlays, crowns & veneers or placing composite restorations. Used for cleaning prior to bonding orthodontic brackets.

COMPOSITION:

Abrasive Powder (less than 65µm), Mint Flavor

INSTRUCTION FOR USE:

1-Before opening the cap, shake the bottle up and down with hands for 3-5 times.

2-Open the cap and pour the appropriate amount of powder into the powder tank of the air polishing device according to the patient's oral condition, and then tighten the cap.

3-Activate the air polishing device and polish the required areas with the appropriate nozzle according to the instructions.

PRECAUTIONS:

►Patients suffering from upper respiratory tract infections, from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device.►The jet of air and powder could cause respiratory difficulties.►If allergic reactions are observed, stop using the product and completely remove the powder from mouth.►The product should not be used on restorations, veneers, or bridges, as it may damage these restorations.►Avoid using under the sub-gingival part.

Storage:

►Keep in original packaging in a cool place away from light and moisture.►Keep out of reach of children.►Close the cap after each use to prevent moisture absorption.►Shake the product well before using.

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