## WETTING RESIN

Light Cure Hydrophobic Unfilled Resin

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**Production License No:**  
17339065

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**INTRODUCTION:**

composite materials from sticking to instruments during both direct and indirect restorations. It provides a smooth, glossy surface and allows precise control in shaping composite materials. Additionally, it can be used as an enamel bonding agent in fissure sealants and as a wetting resin for reinforcing polyethylene fibers. Note: This product is not a substitute for dentin bonding agents and should not be used in restorations involving dentin. Due to its hydrophobic nature, it is unsuitable for dentin and requires a hydrophilic bonding agent instead.

**INDICATIONS:**

- Prevents instrument sticking during direct and indirect composite restorations.
- Enamel bonding agent. (e.g., for fissure sealants)
- Bonding agent for porcelain and lithium disilicate veneers.
- Wetting resin for polyethylene fiber reinforcement.

**COMPOSITION:**

Base resin, Diluent, Photoinitiator, Accelerator, Hydrophobic agent.

**INSTRUCTIONS FOR USE:**

1. Dispense one or two drops of the resin into a mixing well.
2. After placing the first composite layer, dip the instrument into the resin and shape the composite as desired.
3. Light cure for 10 seconds using an LED curing light.
4. Repeat as needed until the restoration is complete.
5. Follow the composite manufacturer's curing instructions for full polymerization.
6. For long procedures, use a light-protective cover over the mixing well to prevent premature curing.
7. After use, securely close the syringe cap and store at the recommended temperature.

**PRECAUTIONS:**

▶For dental use only.▶Avoid contact with skin, eyes, or soft tissue.▶In case of contact, wash with soap and water; if irritation persists, seek medical advice.▶In case of eye contact, rinse thoroughly with water and consult an ophthalmologist immediately.▶Do not use after the expiration date.▶Discontinue use if allergic reactions or skin rashes occur and consult a physician.

**STORAGE CONDITIONS:**

▶Store in the original packaging at room temperature.▶Avoid exposure to temperatures above 35°C or below 10°C to prevent phase separation.▶Keep away from direct light & out of reach of children.

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