## CLOREX Resto

Chlorhexidine Antibacterial Agent

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**Production License No:**  
33452629

**INTRODUCTION:**

CLOREX Resto contains 2% Chlorhexidine Gluconate with a balanced pH of 6.5–7.5 and a mild mint fragrance, specially formulated for restorative dental procedures. It is available in solution and gel forms. By inhibiting matrix metalloproteinases (MMPs) enzymes responsible for degradation of dentinal collagen fibers, it helps increase the durability of resin–dentin bonds and enhance the long-term stability of restorations. CLOREX Resto provides antimicrobial decontamination effective against *Enterococcus faecalis* and *Candida albicans*, ensuring a clean and safe dentin surface prior to bonding. Its bio-compatible formulation preserves natural dentin moisture without interfering with resin materials or adhesive systems, allowing for precise and controlled application.

**INDICATIONS:**

- Inhibition of dentin-degrading enzymes to enhance the longevity of resin bonds.
- Disinfection of dentin surfaces prior to adhesive procedures (effective against *E. faecalis* and *C. albicans*).
- Use in composite restorations to improve long-term clinical performance.

**COMPOSITION:**

Chlorhexidine Gluconate 2%, Natural Mint Essence.

**INSTRUCTIONS FOR USE:**

1. After tooth preparation, gently dry the surface (avoid over-drying).
2. Apply an adequate amount of the product to the dentin surface and leave for 30 – 60 seconds.
3. If using CLOREX Gel, rinse the treated area with water and gently dry.
4. Proceed with adhesive application and restorative steps according to manufacturer instructions.

**PRECAUTIONS:**

- ▶For dental use only.▶Avoid contact with eyes and skin. In case of contact, rinse immediately with plenty of water; if irritation persists, seek medical advice.▶If swallowed, drink plenty of water, do not induce vomiting, and seek medical attention immediately.
- ▶Discontinue use if sensitivity or skin irritation occurs and consult a physician.

**STORAGE CONDITIONS:**

- ▶Store in original packaging at room temperature.
- ▶Keep away from direct sunlight and out of reach of children.

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