CANAL DRY

Tooth Tissues Liquid Moisture Eliminator

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Nik Darman Asia
No. 239, Karafarini St.,
Pardis Tech Park, IRAN

Production
License No:
33452629

CANAL Dry Tooth Tissues Liquid Moisture Eliminator

INTRODUCTION:

Nik Darman Canal Dry is a multi-functional solution designed for drying, degreasing, and disinfecting root canals, as well as preparing restorative surfaces. Formulated with fast-evaporating solvents, bio-compatible disinfectants, and hydrophobic silicone, it ensures thorough drying while providing an effective antibacterial effect. The silicone component forms a temporary hydrophobic layer that prevents saliva from re-entering the canal—without adversely affecting the adhesion of sealers or the bonding strength of restorative materials. This solution is also highly effective for removing calcium hydroxide pastes containing iodoform and for conditioning restorative surfaces prior to bonding.

INDICATIONS:

1. Complete drying of root canals before obturation.
2. Prevention of recontamination during temporary endodontic treatments.
3. Enhancement of sealer and bonding performance in endodontic therapy.
4. Removal of calcium hydroxide pastes containing iodoform.
5. Prevention of saliva re-entry during endodontic procedures.
6. Surface cleaning and degreasing prior to composite restorations.
7. Disinfection of prepared surfaces in restorative dentistry

COMPOSITIONS:

Pharmaceutical-grade Ethanol, Stabilizer, Bio-compatible Disinfectant, Peppermint essence.

INSTRUCTIONS FOR USE:

1. Using a dropper or a soaked paper point, apply 1–2 drops of Canal Dry into the canal or onto the restorative surface.
2. Allow the solution to evaporate naturally, or gently dry with an air syringe (no pressure).
3. Repeat if necessary until the area is completely dry and free of grease.
4. Proceed with obturation, bonding, or placement of the restoration.

PRECAUTIONS:

▶For dental use only. ▶Avoid contact with eyes & skin. ▶If swallowed, drink plenty of water, do not induce vomiting & seek medical attention immediately. ▶Keep away from heat, flame & ignition sources.

STORAGE:

- ▶Store in the original container at room temperature.
- ▶Protect from light, moisture and keep out of reach of children

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